

Headband for Quha Zono X

User Manual



Headband for Quha Zono X

Headband for Quha Zono X is a way to wear the Quha Zono X mouse on the user's head. It contains an adjustable headband and a magnetic fastening holder for Quha Zono X. The headband width is adjustable from S to XL.

Headband for Quha Zono X has a 3.5 mm external switch connector. An optional external switch can be used for left mouse clicks.



Quha Zono X mouse

Installation

Headband for Quha Zono X is delivered in two parts: the headband and the magnetic holder.

To assemble your Headband for Quha Zono X, push the headband into the connector on the magnetic holder until it is fully inserted. Make sure the holder is inserted in the correct way as shown in the picture on page 2.

The width of the headband can be adjusted to fit the user.

You can attach your Quha Zono X mouse to the magnetic holder either before or after placing the Headband on your head. You can turn the mouse on either before or after attaching it to the holder, whichever is more convenient for you.

To attach the Quha Zono X mouse to the magnetic holder, bring it close to the holder and the magnets will grab it into the correct position automatically. The mouse stays firmly in place and is also easy to detach from the holder.







External Switch

An external switch with a 3.5 mm plug can be inserted to headband for left mouse clicks. Connect the plug to the connector on the magnetic holder.



Usage

When wearing Quha Zono X with the Headband, make sure the Quha Zono X mouse is located on the right side of the users' head.



Compatibility

Headband for Quha Zono X is compatible with the Quha Zono X mouse.

Keeping Headband for Quha Zono X Clean

Headband for Quha Zono X is easy to keep clean. The plastic parts of the headband can be wiped with a moist cloth and mild detergent.

European Compliance

The manufacturer of this product confirms that this product complies with the European Union regulations for safety, health, environmental and consumer protection.

UKCA

UK RESPONSIBLE PERSON AVENTIDO Limited, Bridgewater House, Century Park, Caspian Road, Altrincham, Cheshire, WA14 5HH, United Kingdom

FCC Compliance

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Safety

This device contain magnets. Do not place it close to magnetically sensitive devices such as pacemakers, hearing devices, credit cards, mechanical watches, data storage media etc. as they may get damaged.

Product Disposal

When the device has reached its end-of-life it should be recycled as electric and electronic waste.

s product safety,

UK CA



F





Limited Warranty

Warranty on the product is 2 years from the date of purchase. You can find the warranty card in the sales box. See the entire warranty text on our website (<u>www.quha.com/warranty</u>).

© 2023 Quha oy. Quha oy is not responsible for possible errors in this user manual. Quha oy reserves the right to revise the content of this user manual.



Designed and manufactured in Finland by Quha.

www.quha.com

Copyright 2023 Quha oy.

Quha, the Q-mark and Quha Zono X are trademarks or registered trademarks of Quha oy.