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GB User Manual IR-1S Wrist transmitter


Introduction

IR-1S Wrist transmitter is a single channel IR transmitter with internal setting to one of seven different channels. IR-1S Wrist transmitter is a part of the Gewa "Smart House" concept and can therefore be used together with the complete range.

General information about IR-1S Wrist transmitter

On delivery the IR-1S Wrist transmitter is factory programmed to channel 16 or channel 35. The transmitter with a yellow button is programmed on channel 16 and the transmitter with a black button is programmed on channel 35.


Fig 1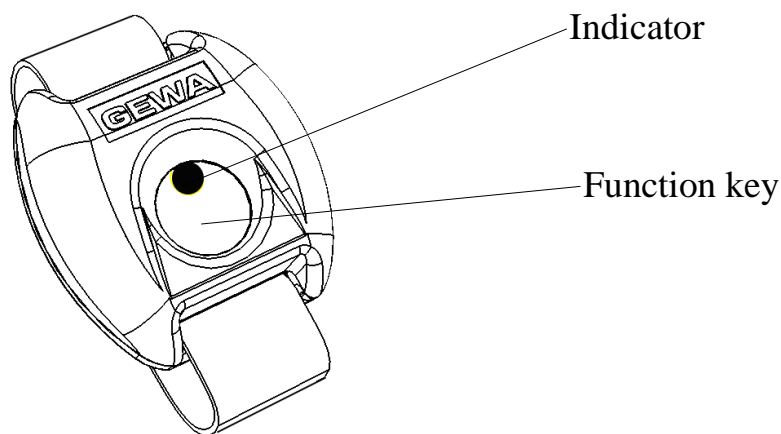

Inserting batteries

Unscrew the screws holding the bottom of the IR-1SP Wrist6 transmitter using a Phillips screwdriver. Remove the bottom and insert three 1.5V silver oxide batteries type SR44. Insert the batteries according to the markings on the bottom of the battery compartment. It is important to use specified batteries for maximum transmission range. Be sure the packing that seals the transmitter is correctly inserted. There will be a battery warning when 75% of the battery power has been used. The indicator (below the function key) will then flash with an interval of one second. The transmitter has enough power to transmit for a while longer.

Fig 2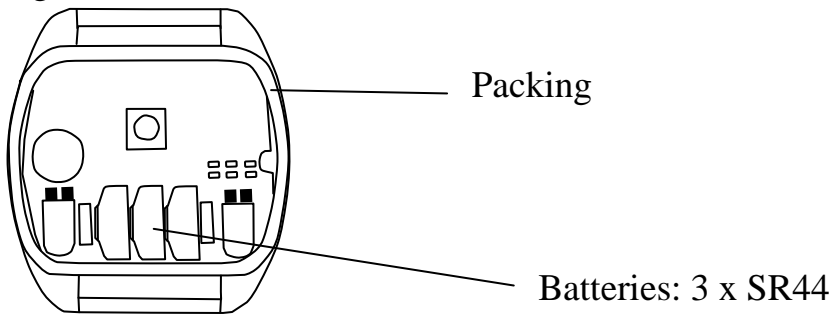


Channel setting

Channel settings are made by scraping or connecting joints between soldering pads on the circuit board. These spots are marked A, B and C in fig 3. The table shows which channels can be used and which joints should be scraped off or connected.

Channel	Joints A	Joints B	Joints C
16	■ ■	■ ■	■ ■
17	■ ■	■ ■	■ ■
18	■ ■	■ ■	■ ■
19	■ ■	■ ■	■ ■
20	■ ■	■ ■	■ ■
4096 code (random)	■ ■	■ ■	■ ■
35	■ ■	■ ■	■ ■

Fig 3

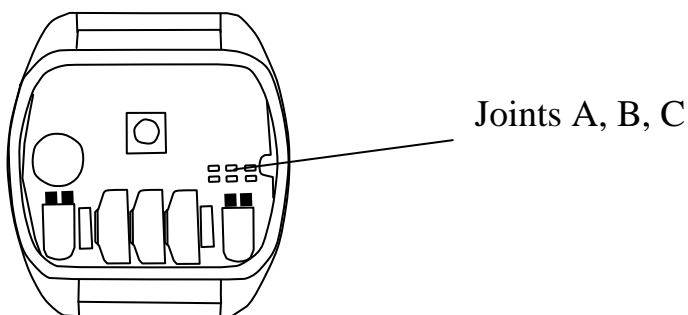How to wear the transmitter

IR-1S Wrist transmitter can be used either with the enclosed wrist strap or hanging on a strap around the neck.

Note! The neck strap will only come with the transmitter with a yellow function key.

Warning!

The string lock is a safety detail for the prevention of strangling. Check that the neck strap ends can easily be drawn out if the transmitter gets caught on something. Never make knots on the neck strap!

Cleaning

To clean the transmitter, wet a cloth in a weak solution of washing up liquid and water, wring out thoroughly and wipe.

Technical Data

Power consumption (stand-by):	Approx. 10 μ A
Power consumption (transmitting):	Approx. 10 mA
Battery life:	Approx. 1 year (transmission time approx. 30 seconds/day)
Key force:	Approx. 1.7 N (170 gr.)
Range:	10-15 m
Measurements (L x W x H):	46 x 44 x 14 mm
Weight (batteries included):	25 gr.
Operating temperature (not valid for batteries):	-20° – +45°C

Conformity with the Medical Devices Directive

IR-1SP Micro is marketed as a technical aid for people with disabilities and meets the requirements in Medical Devices Directive 93/42/EEC including relevant EMC and electrical safety requirements.

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Wir als Hersteller: Abilia AB
Abilia AB, Kung Hans väg 3, Sollentuna, Box 92, 191 22 Sollentuna, Sweden