Gewa Duocom

SE, GB, DE, NO, DK, FR, NL

Gewa AB

Box 92, SE-191 22 SOLLENTUNA, SWEDEN
TEL: +46 (0)8-594 694 00 • FAX: +46 (0)8-594 694 19
E-MAIL: info@gewa.se • WEB: www.gewa.se

SE BRUKSANVISNING FÖR DUOCOM	3
GB HOW TO USE DUOCOM	5
DE GEBRAUCHSANWEISUNG FÜR DUOCOM	7
NO BRUKERVEILEDNING FOR DUOCOM	9
DK SÅDAN ANVENDER DU DUOCOM	11
FR MODE D'EMPLOI DU DUOCOM	13
NL GEBRUIK VAN DE GEWA DUOCOM	15

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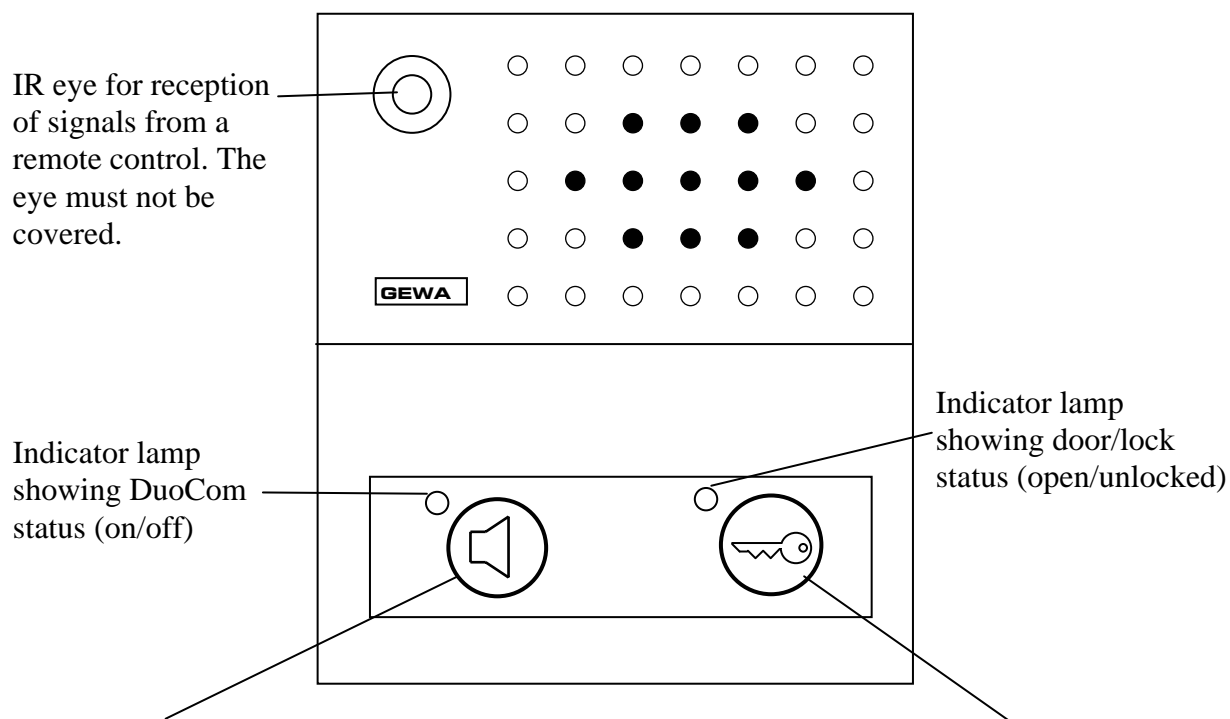
GB How to Use DuoCom

The figure shows the functions for DuoCom room units.

SPEECH BUTTON

- Answer Door

When Using recorded messages.

- Press once: DuoCom starts. Message 1 is activated consisting of the phrase: Hello, who's there?
- Press a second time: Message 2 is activated consisting of the phrase: Please can you call back later?

* The phrases are only examples.

Cleaning

Use only a damp rag with a small amount of washing liquid.

OPENING BUTTON

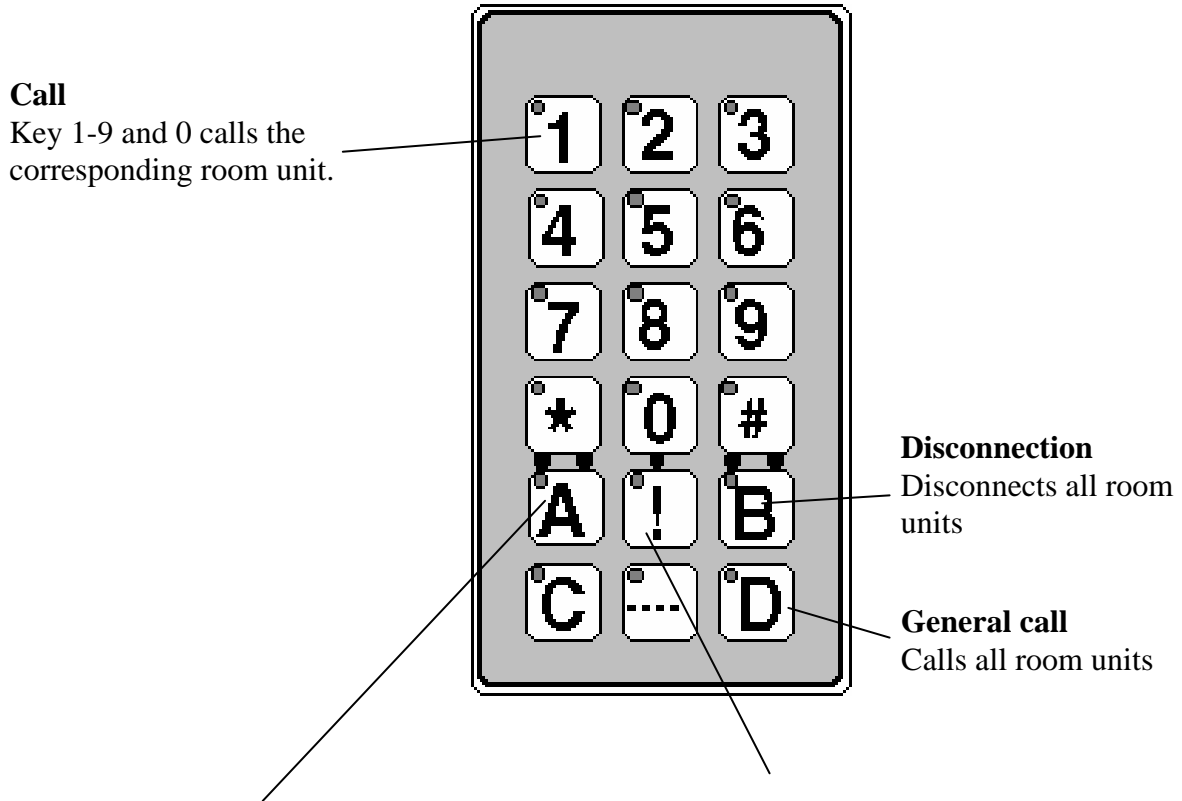
- Lock Release

With recorded messages and automatic door openers connected.

- **Short push:** Opening of lock. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!
- **Long push:** Opening of lock and activation of automatic door opener. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!

How to Use Intercom Telephone

If you want to use DuoCom as an intercom telephone an IR transmitter is required. The figure shows the functions for DuoCom as an intercom telephone.

SPEECH BUTTON

- Answer Door

When Using recorded messages.

- Press once: DuoCom starts. Message 1 is activated consisting of the phrase: Hello, who's there?
- Press a second time: Message 2 is activated consisting of the phrase: Please can you call back later?

* The phrases are only examples.

OPENING BUTTON

- Lock Release

With recorded messages and automatic door opener connected.

- **Short push:** Opening of lock. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!
- **Long push:** Opening of lock and automatic door opener. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!

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