Manual

# SE Control Omni Användarmanual

## GB Control Omni User Guide

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<b>ENGLISH.....</b>	<b>18</b>
DESCRIPTION OF CONTROL OMNI.....	19
USAGE.....	19
EDUCATION.....	19
IMPORTANT INFORMATION .....	20
SYMBOLS .....	21
SURVEY.....	22
CHARGING BATTERIES.....	23
<i>Battery status</i> .....	23
SWITCH ON/OFF.....	24
WAKE UP SCREEN.....	24
ALERT .....	24
<i>Indicator lamp for Alert</i> .....	25
<i>Alert button on the screen</i> .....	25
<i>Alert button</i> .....	25
<i>Alert function with control switch</i> .....	25
SIM-CARD .....	25
MEMORY CARD .....	25
MAKING PHONE CALLS.....	26
<i>Answering</i> .....	26
<i>Hang up</i> .....	26
<i>Making phone calls</i> .....	27
<i>Sending SMS</i> .....	28
<i>Add contacts in the phone book</i> .....	29
CONNECTING HEADSET.....	29
CONNECTING CONTROL SWITCHES .....	30
INDICATING IR/RADIO .....	30
SMS STATUS .....	30
STYLUS .....	30
USB .....	30
RUBBER PROTECTION .....	30
BACK UP.....	31
<i>Resetting</i> .....	31
DOCKING STATION .....	31
RECYCLING WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT.....	31
ENVIRONMENT .....	31
CLEANING .....	32
ACCESSORIES INCLUDED .....	33
TECHNICAL DATA.....	33
THE MEDICAL DEVICE DIRECTIVE .....	36

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# English

## ***Description of Control Omni***

This manual contains important information how to use and handle Control Omni. To make settings read the manual Control Omni Manual Settings.

## ***Usage***

Control Omni is an advanced remote control with built-in mobile phone that has been developed to be used by persons with disabilities. Control Omni is controlled by direct touch on screen but it is also possible to connect control switches with scanning function. Control Omni is programmable which means that IR-codes can be learnt from other remote controls like TV, video and stereo, devices that thus can be controlled. Control Omni is equipped with a built-in radio transmitter for environmental control. The user can make most settings himself by touching the screen but also by scanning. The screen is dynamic meaning that what is shown on the screen also can be arranged to the users needs and preferences.

Control Omni is developed to be used mainly at home but also at rehabilitation clinics or other nursery homes.

## ***Education***

The installation engineer or occupational therapists have the responsibility to provide the user with necessary information to make sure the user will be able to use Control Omni in the right way

Family members as well as nursing staff should also have information about how the product works.

## **Important information**

For a safe use of Control Omni please consider the following:

<p>Electrodes used on the skin or invasive control switches must not be connected to Control Omni. Use only CE-marked control switches.</p>
<p>Use only the enclosed battery charger type: GEW100. No other power supply or battery charger may be connected. Connecting wrong battery charger may damage the electronics and batteries and at worse cause fire.</p>
<p>Batteries must only be changed by Abilia AB or by Abilia authorized workshops.</p>
<p>Service or repairs must only be made by Abilia or by Abilia authorized workshops.</p>
<p>Control Omni or Control Omni holders must not be modified.</p>
<p>If the product has been damaged send it to Abilia AB for repair.</p>
<p>Control Omni must not be used in environment rich in oxygen.</p>
<p>Control Omni contains different kinds of radio transmitters (GSM-telephone) which might disturb other sensitive equipment e.g. operating rooms, intensive care, at home with intensive care equipment or in aeroplanes. Radio transmitters in Control Omni must be switched off in those environments. Follow the local regulations.</p>
<p>If the built-in mobile phone is not correctly used you might get a loud ring signal directly into your ear. This might happen if Control Omni is turned in the wrong direction and at the same time kept close to the ear.</p>
<p>Control Omni must only be used in a dry environment 0-40°C and only charged between 0-30°C. Damp might damage the circuit board and the batteries and cause fire.</p>
<p>Touchable parts of Control Omni might exceed 41°C if it is exposed to direct sunlight.</p>
<p>Do not use any disinfection but "Alsol alcohol chlorhexidine or ethyl alcohol. These liquids are inflammable. Should be handled with care. See chapter "Cleaning".</p>

When using "Alert" you must be aware of the fact that batteries might run out, connected plugs might be released unintentionally, difficulties for the user to reach the switches, the telephone network is down, software problems etc. Alert must not be used as emergency calls and never be used if a person is in danger when a Alert fails.


The only way to disconnect the power to Control Omni when charging is to remove the charger from the wall outlet.

## Symbols

	Important information. Must be read.		Recycling Waste Electrical and Electronic Equipment
	Alert and button for Alert		SMS
	IR/radio transmission		Battery indicator
	Answer calls		Hang up
	Wake up screen		

# Survey

Fig. 1

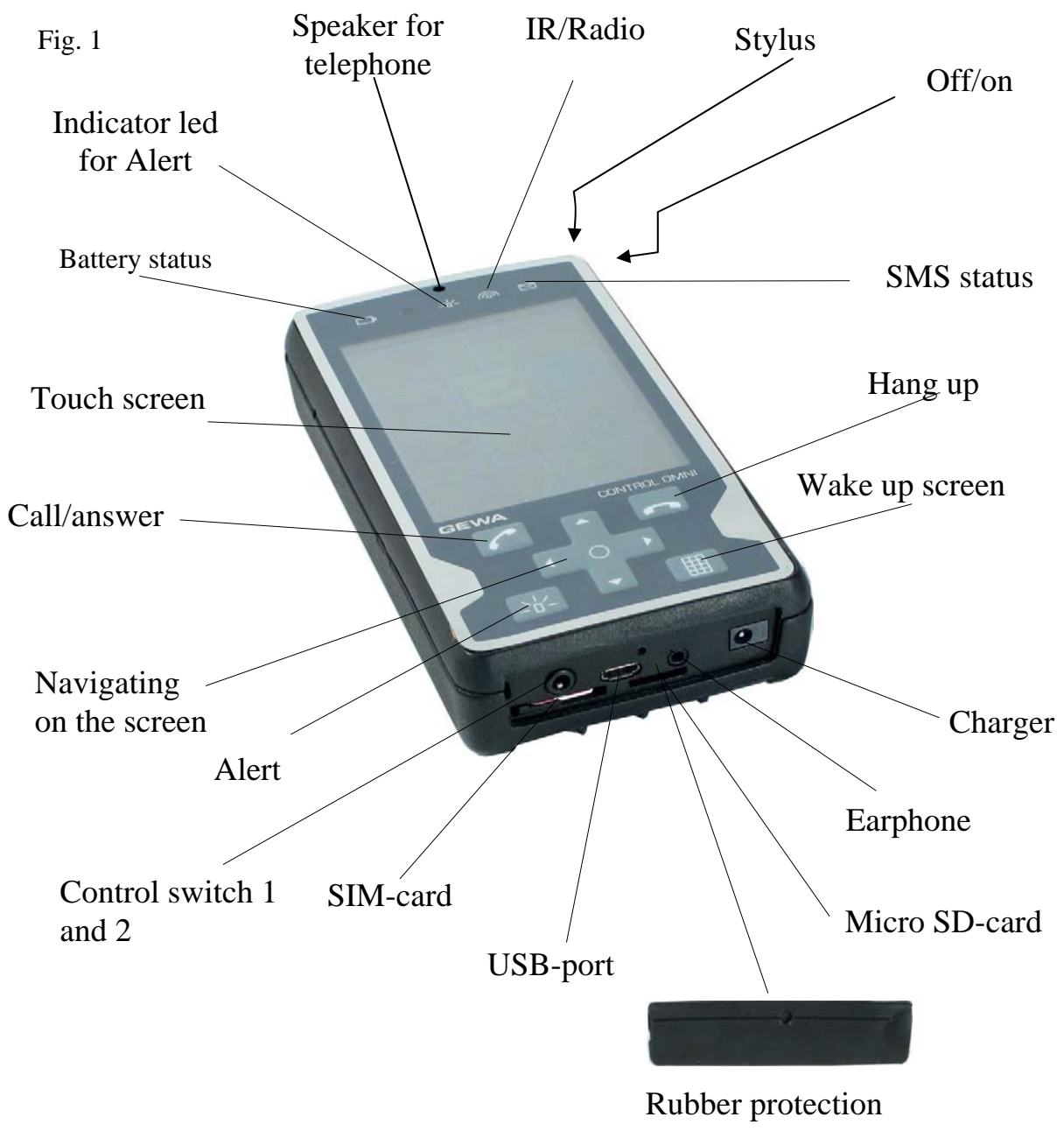## Charging batteries

Control Omni must be charged at regular intervals. How often this must be made depends on the time Control Omni is used.

Control Omni starts warning with a beep when approx. 25 % of the battery capacity remains and you, at the same time, activate the screen . Next warning will occur when approx. 7 % of the battery capacity remains. If it is not possible to charge at once, limit the use to save batteries.

Note! Charge the batteries regularly once a day and in time before the batteries run out. Do not let the batteries run out completely. Switch off Control Omni if it is not to be used for some time.

The charger can be connected continuously.

The only way to disconnect the power to Control Omni when charging is to remove the charger from the wall outlet. If the mobile phone or Gewa Radio (GR-unit) is not used, battery capacity can be saved by turning these units off.

### Warning!

Only use the enclosed battery charger type GEW100. Connecting the wrong battery charger might damage electronics and batteries and at worse cause fire.

Batteries must only be changed by Abilia or by Abilia authorized working shop.

## Battery status

Control Omni shows the battery status in different ways:

- A symbol on front above the screen (A).



Flashes orange when the battery needs charging (approx. 7 % left).

Flashes green light when charging.

Green light when the battery is fully charged.

- Battery indicator on the page for telephone function (B).

- Specific button (C) that can be put on either page showing the battery status.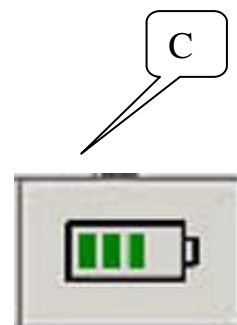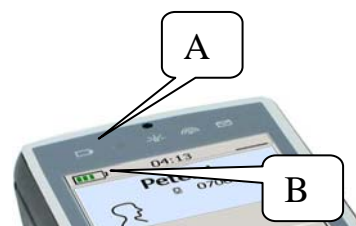


## Switch on/off

If Control Omni is not to be used for a while it can be switched off. Everything that has been programmed is saved in the memory. It is still possible to charge.

Switch on: Press the stylus into the hole once (A)

Switch off: Press and keep the stylus pressed for a few seconds.

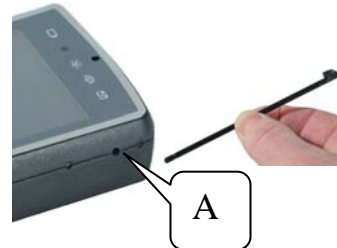## Wake up screen

The screen automatically goes down after 30 sec when not used. There are three ways to wake up the screen:

- Press directly on the screen with your finger or by a stylus (A). Note! Do not use sharp objects as it might damage the display.
- Press a specific button (B) below the screen.
- Press an external control switch (C).

## Alert

”Alert” is used when you want to attract somebody’s attention that you need assistance.

Control Omni can activate Alert in different ways:

- transmit an IR-signal which activates a relay in an IR receiver.
- transmit an RF-signal which activates a relay in a radio receiver.
- send an SMS with the built-in telephone.
- Call a telephone number with the built-in telephone.
- Activate the relay output.

Note! Several Alert functions can be activated simultaneously. To set different Alert functions see manual Control Omni Manual Settings.

<p><b>Warning!</b>          When using "Alert" you must be aware of the fact that batteries might run out, connected plugs might be released unintentionally, difficulties for the user to reach the switches, the telephone network is down etc. Alert must not be used as emergency calls and never be used if a person is in danger when Alert fails.</p>	
--	---

**Indicator lamp for Alert**

Indicator lamp (A) flashes when call Alert been initiated.

**Alert button on the screen**

It is possible to put Alert button (B) on the screen. button can be placed on one page or on several different pages.

**Alert button**

(C) This button is the one that works longest when the battery is running out.

**Alert function with control switch**

By keeping the external control switch (D) pressed Alert will be activated. The time is set from 1-10 sec. or switched off completely; see manual Control Omni Manual Settings. When activating Alert a symbol with a beep will appear on the screen.


The

**SIM-card**

If you want to be able to call with a mobile phone or send SMS Control Omni must contain a SIM-card. A mobile phone subscription is not included.

**Memory card**

Always keep the memory card in Control Omni (B). When making a backup everything is saved on the memory card.

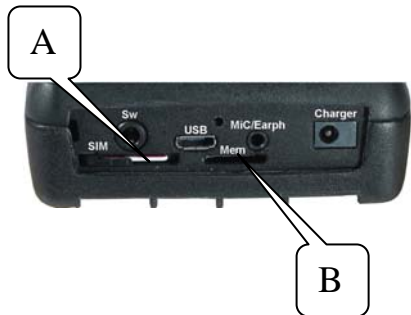## ***Making phone calls***

### **Answering**

When you receive a phone call answering can be made as follows:

- Press the answer button (A) shown on the screen.
- Press the answer button (B) below the screen.
- Press the external control switch (C) and step to the answer button.

### **Hang up**


When you want to end a conversation you can do as follows:

- Press on the screen (A) to hang up.
- Push the button (B) under the screen
- Push the external control switch (C) to step to the button to hang up.


## Making phone calls

Do as follows to make a call; press directly on the screen or step forward by the external control switch: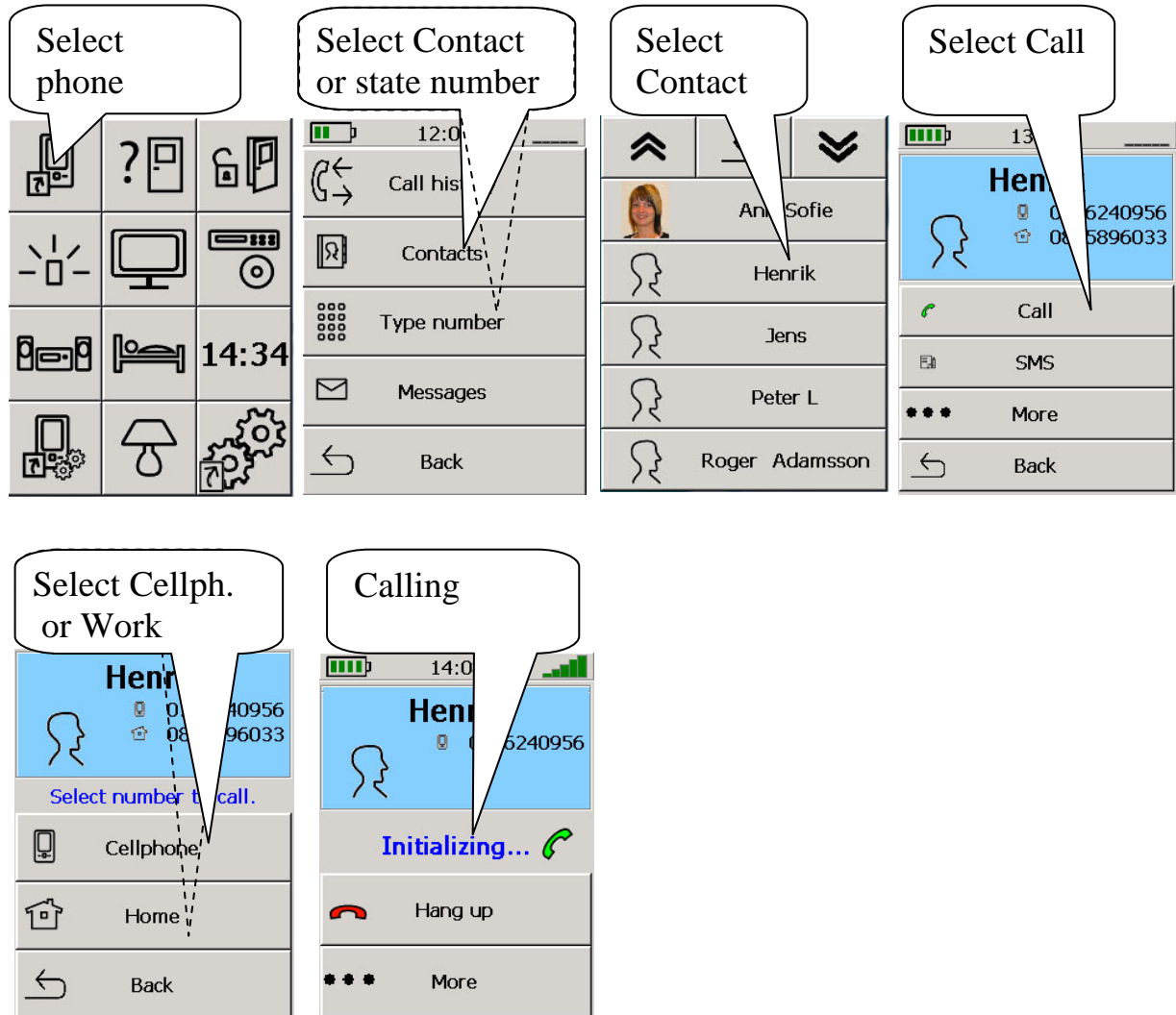


## Sending SMS

Do as follows to send SMS; press directly on the screen or step forward by external control switch.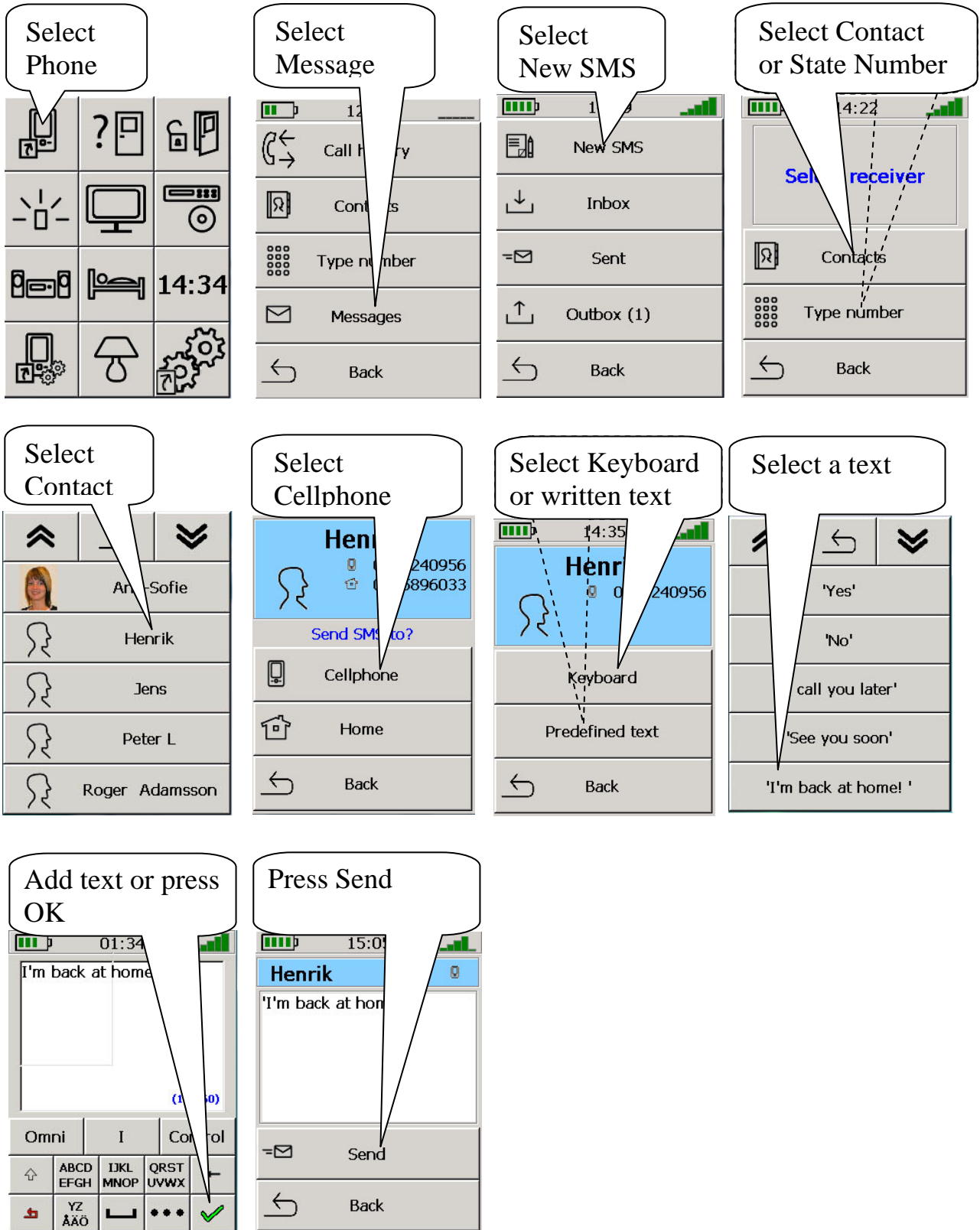

## Add contacts in the phone book

Do as follows to add contacts into phone book (contacts)

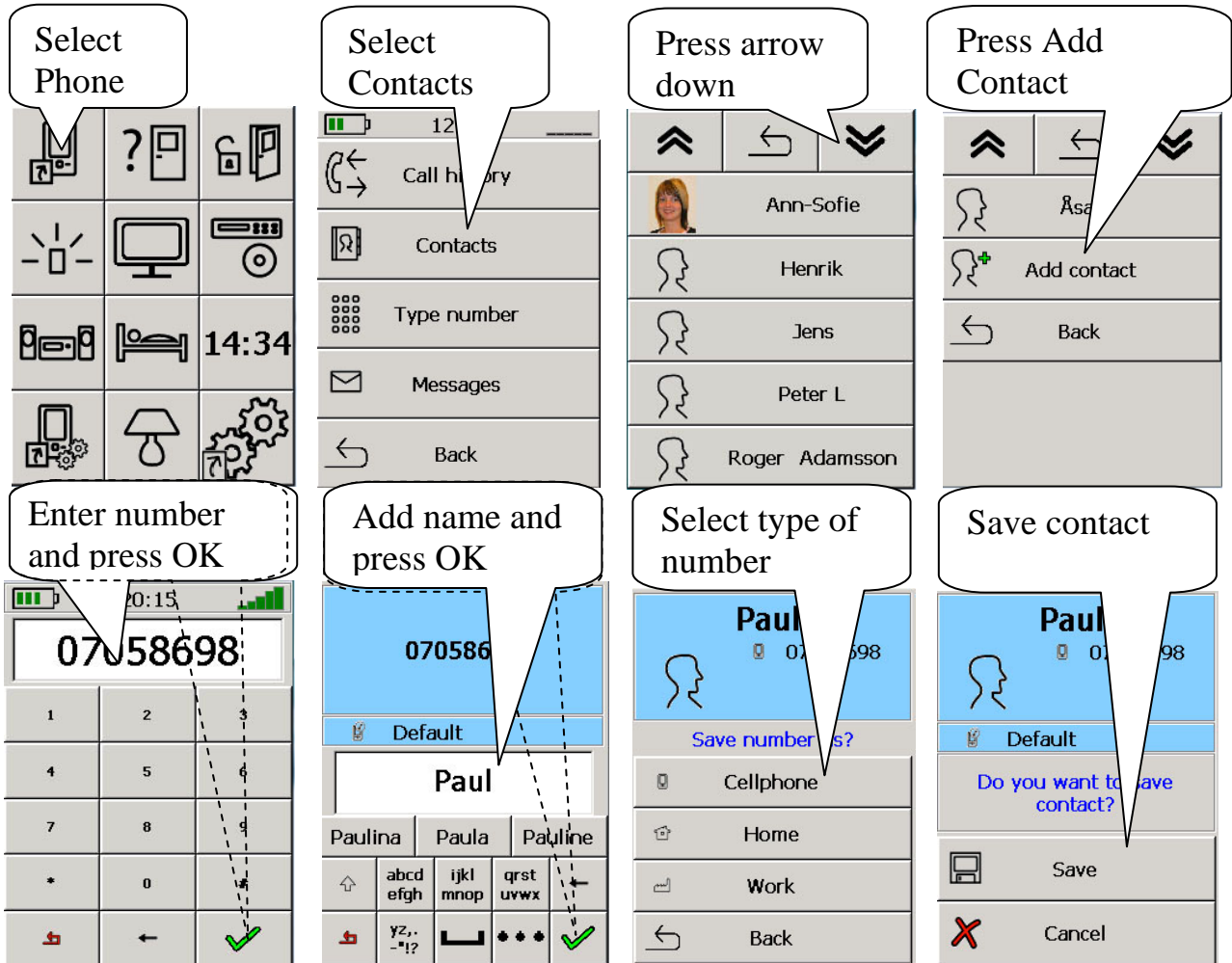## Connecting Headset

If you are not able to hold Control Omni in your hand when calling, connect the enclosed Headset

## Connecting control switches

Control Omni has an input for external control switches. Use a connecting cable with 3,5 mm jack plug. To connect 2 control switches a split cable is needed. It is not included but must be ordered separately (Item no: 429050).


### Warning!

Electrodes used on the skin or invasive control switches must not be connected to Control Omni. Use only CE-marked control switches.

## Indicating IR/Radio

When transmitting an IR-signal or radio signal to control an object, the indicator symbol is flashing (A).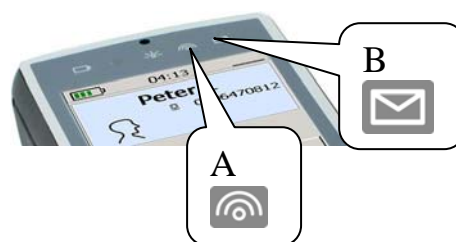

## SMS status

Indicates when a SMS has been received (B).

## Stylus

Remove the stylus from its compartment (A). The stylus can be used to point at the screen. You can also use your finger to point directly on the screen. The stylus can also be used when Switching off/Switching on Control Omni.

## USB

USB input is used for connecting to computer (A).


## Rubber protection

To protect the inputs against damp and dust it is important that a rubber protection covers the inputs when they are not used. It is possible to make a hole in the rubber protection to be able to connect both a control switch and a headset.

## ***Back up***

When different pages and different settings are made its important to make a back up. Before it is made check that the enclosed memory card is pushed into the compartment on the back of Control Omni (A). If there already is a backup made on the memory card that file will be erased. Make sure that the memory card always is inserted in the compartment. See chapter in the Control Omni Manual Settings how to make a backup.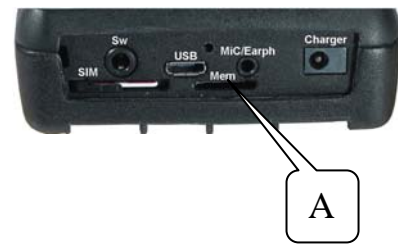


## ***Resetting***

If, for any reason, Control Omni should break it is possible to recreate the settings made if a back up has been made. See chapter in the Control Omni Manual Settings how to make settings.

## ***Docking station***

Control Omni can be mounted on a docking station (A) which can be fixed to different kinds of stands (B). This makes it possible to attach Control Omni to the bed or to the wheelchair. When the docking station is used, charger and control switches shall be connected to the outlets on the docking station.

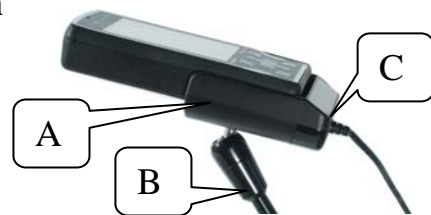## ***Recycling Waste Electrical and Electronic Equipment.***

Control Omni contains electronics components that must be recycled. When the product is of no use, hand it in to a recycling centre for waste electrical and electronic equipment.

## ***Environment***

Control Omni must only be used in a dry environment 0-40°C and only charged between 0-30°C. Damp might damage Control Omni and at worse cause fire. Make sure that Control Omni is not exposed to damp.

## **Cleaning**

Control Omni can be cleaned with a damp cotton-wool pad med Alsol alcoholic, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid).  
Wipe dry after cleaning.

<p><b>Warning!</b> Do not use any disinfectant but Alsolsprit, Chlorhexidine or ethyl alcohol.  These liquids are inflammable, handle with care!</p>	
--	---

## ***Accessories included***

<b>Item no</b>	<b>Denomination</b>
429000	Control Omni
429020	Control Omni Charger GEW100
429035	Control Omni Headset
429030	Control Omni Micro SD Card incl. USB adapter
429025	Control Omni Stylus
429045	Control Omni Rubber protection
890410	Control Omni Safety strap
809210	Control Omni User guide SE/GB
809220	Control Omni Manual settings SE
809220	Control Omni Manual settings GB

## ***Technical data***

Radio system:	Gewa RS
Mobile phone:	GSM 900/1800
Operating force screen:	<150 gr
Operating force buttons:	<150 gr
Range IR:	10-30 m
Range Gewa Radio System:	10-20 m
Measurements excl. docking station (L x W x H):	140 x 76 x 27 mm
Weight excl. charger and docking station:	0,3 kg
Operating temperature (incl batteries):	0 till +40° C
Operating temperature when charging:	0 till +30° C
Rated voltage charger GEW100:	100-240 V AC/5VDC
Rated current charger GEW100:	600 mA
Rated frequency charger GEW100:	50-60 Hz
IP-Class:	IP-20 (Dry environment)
ROHS-compatible	Yes
Memory card	Micro SD-card
Battery time (Standby)	Approx. 40h
Talk	Approx. 8 hours

## ***The Medical Device Directive***

Control Omni is marketed as a technical aid for persons with disabilities and it satisfies the requirements of the Medical Device Directive 93/42/EEC. This includes, for example, relevant requirements for EMC, for electrical safety and risk management.

**ABILIA**

GEWA AB | FALCK IGEL